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## BEFESTIGUNG PORZELLAN

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# DERBY P-GUARD®

Welcome to your new Gordon Ellis product.  
We hope that it proves a useful aid in your daily life.

**SEE INSIDE AND REVERSE FOR FITTING INSTRUCTIONS**  
AVAILABLE IN THE FOLLOWING LANGUAGES: **EN FR DE**

**EN** ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE MANUFACTURER - CONTACT DETAILS BELOW

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Your **DERBY P-GUARD**® was specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

DESIGNED  
IN THE UK

**GORDON ELLIS & CO.**  
TRENT LANE  
CASTLE DONINGTON  
DERBY, ENGLAND DE74 2AT

EC REP

MADE IN CHINA

**AUTHORISED REPRESENTATIVE**  
ADVENA LTD, TOWER BUSINESS CENTRE,  
2ND FLOOR, TOWER STREET, SWATAR, BKR  
4013 MALTA

MD

IFU-REF | 60341/1020-1

PART REF | 60341

VIEW THE ENTIRE RANGE ONLINE AT  
[GORDONELLISHEALTHCARE.CO.UK](http://GORDONELLISHEALTHCARE.CO.UK)

[f gordonellis.healthcare](https://www.gordonellis.healthcare)  
[@EllisHealthcare](https://twitter.com/EllisHealthcare)

VIEW THE ENTIRE RANGE ONLINE AT  
[GORDONELLISHEALTHCARE.CO.UK](http://GORDONELLISHEALTHCARE.CO.UK)

[f gordonellis.healthcare](https://www.gordonellis.healthcare)  
[@EllisHealthcare](https://twitter.com/EllisHealthcare)

## INSTRUCTIONS FOR USE

### FIXING TO PORCELAIN

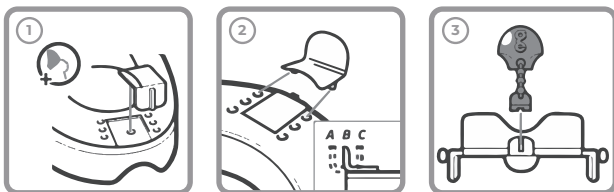
LOCATE DESIRED POSITION FOR THE **P-GUARD**<sup>®</sup>. THOROUGHLY CLEAN AREA WITH ALCOHOL WIPE. DETACH FIXING PLATE FROM **P-GUARD**<sup>®</sup>. REMOVE PAPER FROM ADHESIVE PAD ON THE FIXING PLATE. STICK FIXING PLATE TO PORCELAIN, WAIT 72 HOURS.

**DO NOT ATTEMPT TO SCREW OR DRILL INTO PORCELAIN.**

RE-ATTACH **P-GUARD**<sup>®</sup> TO FIXING PLATE.

### FIXING TO SERENITY RAISED TOILET SEAT

### FIXING TO OTHER RAISED TOILET SEATS

1. STICK FIXING PLATE TO RAISED TOILET SEAT.
2. SECURE WITH 2 OR 4 SCREWS AS DESIRED. PUSH SCREWS THROUGH BLANKED OFF HOLES.
3. RE-ATTACH **P-GUARD**<sup>®</sup> TO FIXING PLATE.

**P-GUARD**<sup>®</sup> CAN BE POSITIONED FOR EACH USE USING THE FLEXIBLE LINKS

**P-GUARD**<sup>®</sup> CAN BE REMOVED FROM ITS FIXING PLATE FOR EASE OF ACCESS AND FOR CLEANING.

PH 5 - 9

CLEAN WITH NORMAL HOUSEHOLD CLEANER OR DISINFECTANT.

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