DERBY ELEPHANT FEET®

FURNITURE RAISERS

Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.

SEE REVERSE FOR FITTING INSTRUCTIONS
AVAILABLE IN THE FOLLOWING LANGUAGES:
EN FR DE ES NL

MAX
500 Kg
1,100 lbs

MAXIMUM LOAD PER SET OF 4	MAXIMALES GEWICHT PRO 4ER-SET
POIDS MAXIMUM PAR LOT DE 4	MAXIMAAL GEWICHT PER SET VAN 4
	PESO MÁXIMO POR JUEGO DE 4

EN ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE MANUFACTURER - CONTACT DETAILS BELOW
FR TOUT ACCIDENT GRAVE SURVENANT PENDANT L'UTILISATION DOIT ÊTRE SIGNALÉ AU EC REP - COORDONNÉES CI-DESSOUS
DE JEDER SCHWERE UNFALL WÄHREND DES GEBRAUCHS MUSS EC REP GEMELDET WERDEN - KONTAKTDATEN UNTEN
ES CUALQUIER ACCIDENTE GRAVE QUE OCURRA DURANTE EL USO DEBE INFORMARSE A EC REP - DETALLES DE CONTACTO A CONTINUACIÓN
NL ELK ERNSTIG ONGEVAL TIJDENS HET GEBRUIK MOET WORDEN GEMELD AAN EC REP - CONTACTGEGEVENS HIERONDER

Your ELEPHANT FEET® were specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

9 CM

6070/BK - BLACK
6070 - GREY


14 CM



6074/BK - BLACK
6074 - GREY

9 CM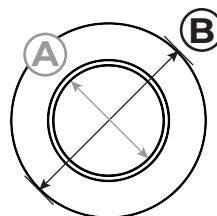

14 CM

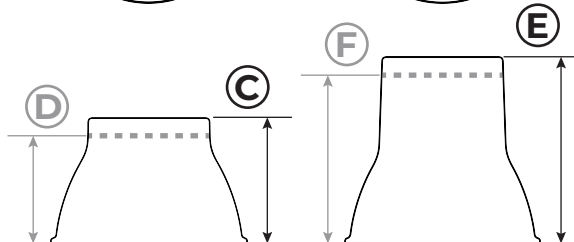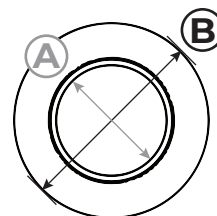A 9 CM Ø*

C 10 CM

E 15 CM

B 16 CM Ø

D 9 CM

F 14 CM

MAXIMUM LEG DIAMETER
DIAMÈTRE DE JAMBE MAXIMUM
MAXIMALER BEINDURCHMESSER
DIÁMETRO MÁXIMO DE PIERNA
MAXIMALE BEENDIAMETER

8.5 CM Ø

GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY. ENGLAND DE74 2AT

EC REP

AUTHORISED REPRESENTATIVE:
ADVENA LTD. TOWER BUSINESS
CENTRE, 2ND FLR, TOWER STREET,
SWATAR, BKR 4013 MALTA

MD

IFU-REF 6070/0520-2

PART REF 6070 + 6074 + 6070/BK + 6074/BK

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

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- EN**
1. ENSURE THE NON-SLIP PADS ARE IN PLACE ON TOP OF THE ELEPHANT FEET
 2. PLACE TWO ELEPHANT FEET CLOSE TO THE EXISTING FEET OF THE FURNITURE
 3. CAREFULLY LIFT FURNITURE ON TO THE ELEPHANT FEET
-  **BE CAREFUL NOT TO STRAIN YOURSELF WHEN LIFTING**
4. REPEAT AT THE OTHER SIDE OF THE FURNITURE
 5. ENSURE THAT EACH LEG IS CENTRED ON THE ELEPHANT FEET
-  **TEST FOR STABILITY**
6. IF THE FURNITURE IS MOVED, ENSURE THAT IT IS SECURE BY REPEATING THE ABOVE INSTRUCTIONS

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VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

 [gordonellis.healthcare](https://www.gordonellis.healthcare)

 [@EllisHealthcare](https://twitter.com/EllisHealthcare)

UK
CA CE