INSTRUCTIONS FOR USE    MODE D'EMPLOI    GEBRAUCHSANWEISUNG

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**EN** ONLY USE SUREGRIP RAISERS UNDER STATIC FURNITURE. WEIGHT OF FURNITURE SHOULD BE EVENLY DISTRIBUTED. IF KNOCKED CHECK TO ENSURE STABILITY. CHECK FOR STABILITY BEFORE EACH USE. ALWAYS RAISE FURNITURE LEGS BY THE SAME HEIGHT. DO NOT USE ON UNEVEN SURFACES. TAKE CARE TO AVOID ENTRAPMENT OF FINGERS IN ADJUSTMENT SLOTS. **WARNING:** POTENTIAL TRIP HAZARD IF CROSS MEMBERS PROTRUDE OUTSIDE THE FOOTPRINT OF THE FURNITURE.

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**EN**  
If using “Raisers” on a hard floor fit the supplied three adhesive feet.

If furniture feet are 8.5cm or more use “Block Adapter” component, **follow A overleaf**. If furniture feet are less than 8.5cm, or have casters **follow B overleaf**.

To change height of “Raisers” place hand into base and turn to desired height. **Ensure all “Raisers” are the same height.**

If using “Link Arms + Spreader Bar”, first follow A or B, and then **C overleaf**. Ensure “Spreader Bar” is across the width of the furniture **not front to back - as this may cause a trip hazard.**

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## FURNITURE RAISERS

### LINK ARMS + SPREADER BAR

Welcome to your new Gordon Ellis product.  
We hope that it proves a useful aid in your daily life.

**SEE REVERSE FOR FITTING INSTRUCTIONS**  
AVAILABLE IN THE FOLLOWING LANGUAGES:  
**EN FR DE ES NL**

**MAX**  
700 Kg  
1540 lbs

**MAXIMUM LOAD**  
PER SET OF 4

**MAXIMALES GEWICHT**  
PRO 4ER-SET

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**MAXIMAAL GEWICHT**  
PER SET VAN 4

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**EN** ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE MANUFACTURER - CONTACT DETAILS BELOW  
**FR** TOUT ACCIDENT GRAVE SURVENANT PENDANT L'UTILISATION DOIT ÊTRE SIGNALÉ AU **EC REP** - COORDONNÉES CI-DESSOUS  
**DE** JEDER SCHWERE UNFALL WÄHREND DES GEBRAUCHS MUSS EC REP GEMELDET WERDEN - KONTAKTDATEN UNTEN  
**ES** CUALQUIER ACCIDENTE GRAVE QUE OCURRA DURANTE EL USO DEBE INFORMARSE A **EC REP** - DETALLES DE CONTACTO A CONTINUACIÓN  
**NL** ELK ERNSTIG ONGEVAL TIJDENS HET GEBRUIK MOET WORDEN GEMELD AAN **EC REP** - CONTACTGEGEVENS HIERONDER

Your SUREGRIP® RAISERS were specifically developed by our UK Design Team to make furniture raising as simple as possible, and with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

**FURNITURE RAISER**  
60570

**LINK ARM**  
60571

**SPREADER BAR**  
60572

**BLOCK ADAPTER**  
WITH ADHESIVE PAD

**LINK ARM**  
60571

**SPREADER BAR**  
60572

**DESIGNED IN THE UK**

**BHTA**  
British Healthcare Trade Association

**5**  
PP

**RECYCLABLE**

**WASTE DISPOSAL**

**GORDON ELLIS & CO.**  
TRENT LANE  
CASTLE DONINGTON  
DERBY, ENGLAND DE74 2AT

**MADE IN CHINA**

**MD**

**IFU-REF** 60570/0520-2

**PART REF** 60570 + 60571 + 60572

**AUTHORISED REPRESENTATIVE**  
ADVENA LTD, TOWER BUSINESS CENTRE,  
2ND FLOOR, TOWER STREET, SWATAR, BKR  
4013 MALTA

VIEW THE ENTIRE RANGE ONLINE AT **GORDONELLISHEALTHCARE.CO.UK**

gordonellis.healthcare



@EllisHealthcare

VIEW THE ENTIRE RANGE ONLINE AT **GORDONELLISHEALTHCARE.CO.UK**


gordonellis.healthcare

@EllisHealthcare


# A BLOCK ADAPTER WITH ADHESIVE PAD

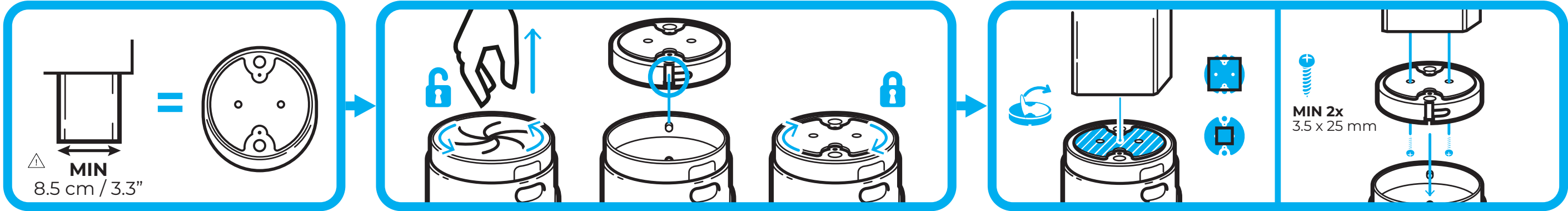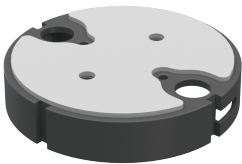# B FURNITURE RAISER 60570
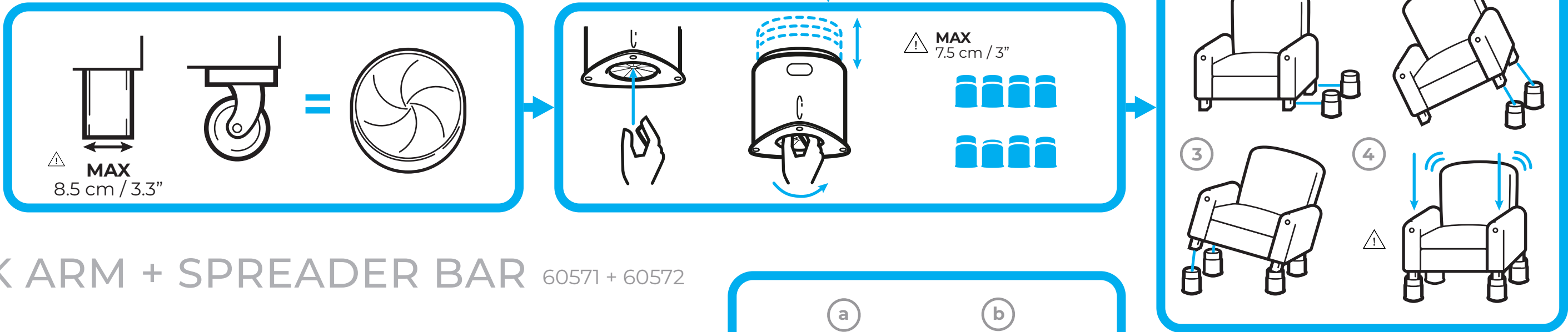
# C LINK ARM + SPREADER BAR 60571 + 60572

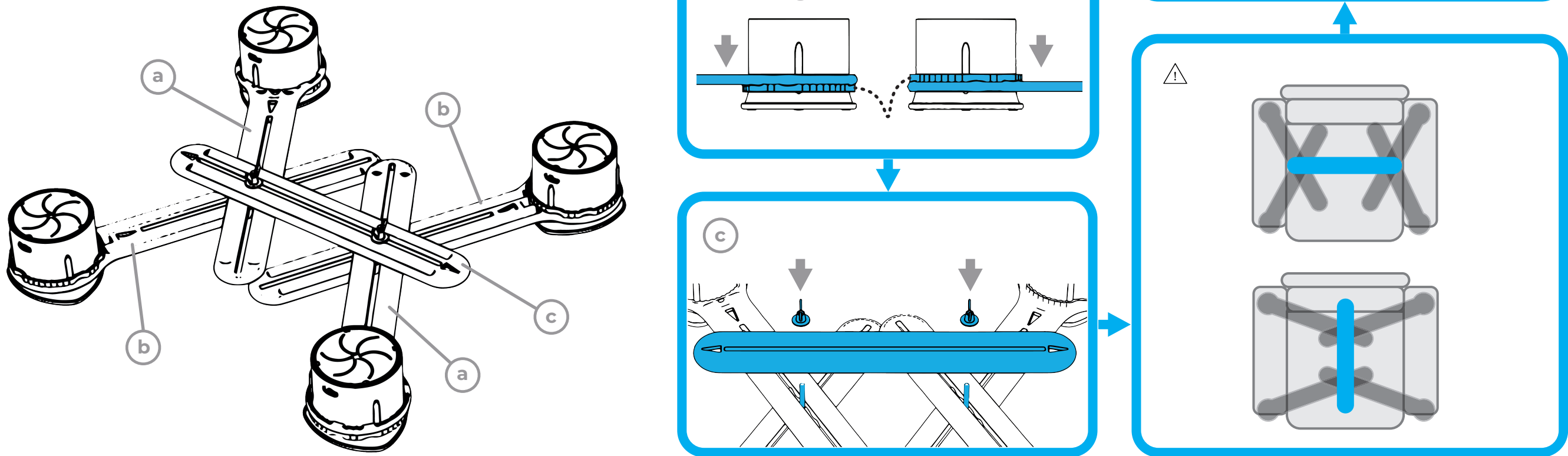FOR FURTHER INFORMATION AND GUIDANCE A VIDEO CAN BE VIEWED BY SCANNING THE QR CODE, OR VISITING:  
POUR PLUS D'INFORMATIONS ET DE CONSEILS, UNE VIDÉO PEUT ÊTRE VISIONNÉE EN SCANNANT LE CODE QR OU EN VISITANT:  
FÜR WEITERE INFORMATIONEN UND ANLEITUNGEN KANN EIN VIDEO DURCH SCANNEN DES QR-CODES ODER DURCH BESUCHEN VON:  
PARA OBTENER MÁS INFORMACIÓN Y ORIENTACIÓN, PUEDE VER UN VIDEO ESCANEANDO EL CÓDIGO QR O VISITANDO:  
VOOR MEER INFORMATIE EN BEGELEIDING KAN EEN VIDEO WORDEN BEKEKEN DOOR DE QR-CODE TE SCANNEN OF DOOR NAAR:

GORDONELLISHEALTHCARE.CO.UK/SUREGRIP-2