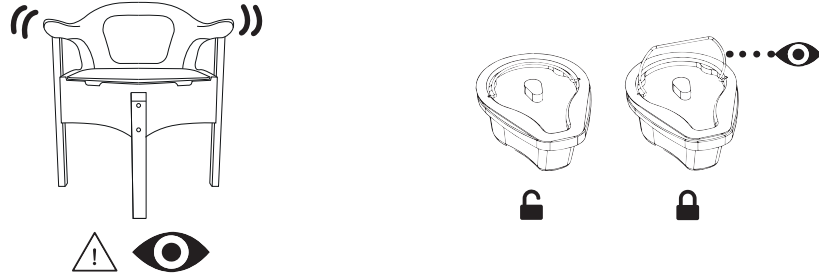
INSTRUCTIONS FOR USE **MODE D'EMPLOI** **GEBRAUCHSANWEISUNG** **ISTRUZIONI PER L'USO**
INSTRUCCIONES DE USO **INSTRUÇÕES DE USO** **GEbruIKSAANWIJZING** **BRUGSANVISNING**
ANVÄNDNINGSSINSTRUKTIONER **INSTRUKSJONER FOR BRUK** **KÄYTTÖOHJEET** **ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ**

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Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.

SEE INSIDE AND REVERSE FOR FITTING INSTRUCTIONS
AVAILABLE IN THE FOLLOWING LANGUAGES:
EN FR DE IT ES NL PT DK SV NO FI EL

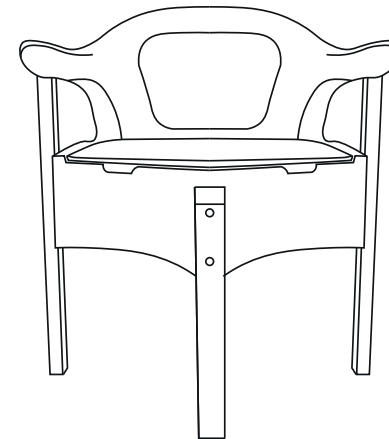
MAXIMUM USER WEIGHT
MAXIMALES BENUTZERGEWICHT
MAXIMUMGEWICHT VAN GEBRUIKER
HÖGSTA ANVÄNDARVIKT
ΑΝΩΤΑΤΟ ΟΠΙΟ ΒΑΡΟΥΣ ΤΟΥ ΧΡΗΣΤΗ

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Your DERBY CORNER COMMUNE was specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY, ENGLAND DE74 2AT

IFU-REF 55085/BR/1022-1

PART REF 55085/BR + 55087/BR

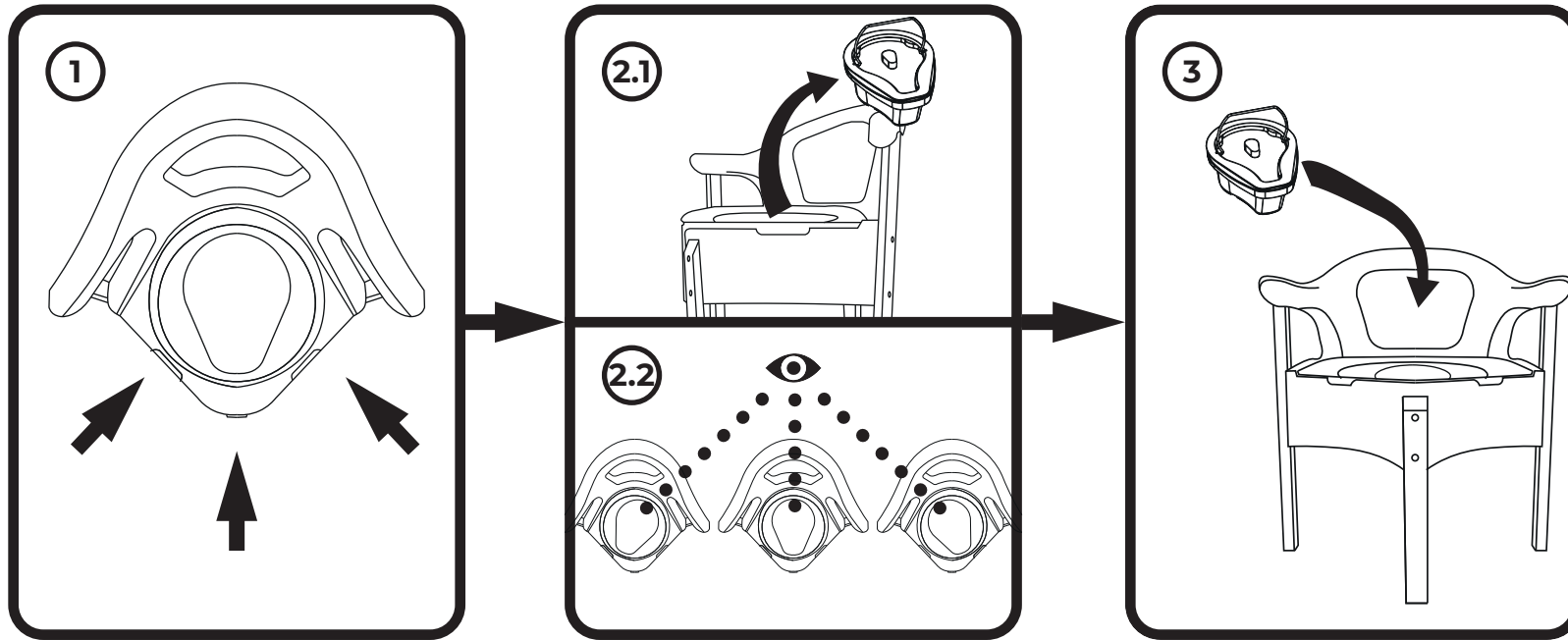
VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

f gordonellis.healthcare
@EllisHealthcare

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

f gordonellis.healthcare
@EllisHealthcare

i FITTING INSTRUCTIONS

EN 1. POSITION THE CHAIR WHERE IT IS COMFORTABLE FOR YOU TO USE. THE CHAIR CAN BE ACCESSED FROM THE FRONT OR SIDE. IT CAN BE SITUATED EITHER IN A CORNER OR BY THE SIDE OF A BED.
2. REMOVE THE SEAT CUSHION AND PAN, AND ROTATE THE PAN HOLDER INTO THE DESIRED POSITION.
3. REPLACE THE PAN. USE THE METAL HANDLE TO TRANSPORT THE CONTENTS TO A DISPOSAL POINT.

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