
LANGHAM MULTI PURPOSE RAISER

Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.

Your **LANGHAM MULTI PURPOSE RAISER** was specifically developed by our UK Design Team who have over 60 years experience in raising furniture, coupled with feedback received from Occupational Therapists. This is the first linked raiser designed to specifically raise chairs, settees, tables and beds equally, with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.


❶ MULTI PURPOSE RAISER

LP/MPR-BK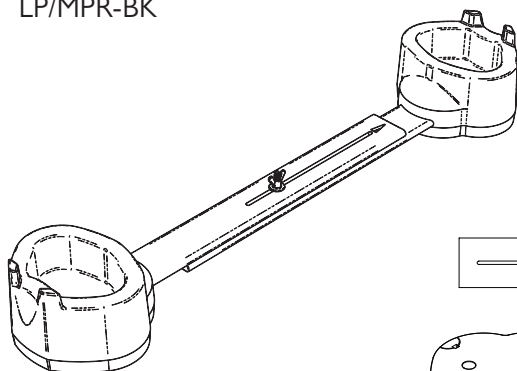

❶ SPREADER BAR

LP/MPR/SB

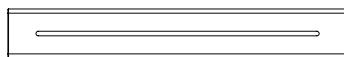
❶ XL SPREADER BAR

LP/MPR/4

❶ CLIP-ON BASE

LP/MPR/BASE-BK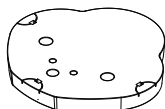

GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY, ENGLAND DE74 2AT

EC REP

AUTHORISED REPRESENTATIVE
ADVENA LTD, TOWER BUSINESS CENTRE, 2ND
FLOOR, TOWER STREET, SWATAR, BKR 4013 MALTA

❶ MD

IFU-REF LP_MPR-BK/0324-1

REF LP/MPR-BK

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK



gordonellis.healthcare

@EllisHealthcare

1 MPR ASSEMBLY INSTRUCTIONS



ARMCHAIRS/ SMALL SOFAS WITH 4 CASTORS/ LEGS/ BLOCKS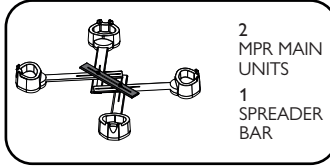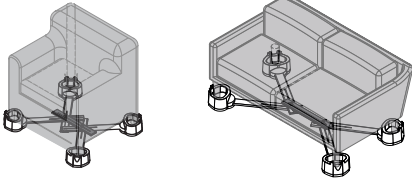

2
MPR MAIN
UNITS
1
SPREADER
BAR

MPR ASSEMBLY INSTRUCTIONS - ARMCHAIRS/ SMALL SOFAS: 1. PLACE THE MPR UNITS ON THE FLOOR BEHIND THE FURNITURE. ATTACH THE SPREADER LINK BAR RUNNING ACROSS THE FURNITURE SIDE TO SIDE. 2. LOOSEN THE WING NUTS. POSITION THE FRONT CORNER SUPPORTS AS FAR OUT AS POSSIBLE. 3. ADJUST THE REAR CORNER SUPPORTS SO THEY ARE THE SAME WIDTH AS THE FURNITURE. 4. TILT THE FURNITURE FORWARDS AND SLIDE EACH OF THE REAR CORNER SUPPORTS UNDER THE REAR CORNERS OF THE FURNITURE. 5. LOWER THE FURNITURE ON TO THE CORNER SUPPORTS ENSURING THE CASTOR/ LEG IS SITTING INSIDE THE SUPPORT WELL OR THE FRAME/ BLOCK LEG IS RESTING ON THE TOP FACE. 6. MOVE TO THE FRONT OF THE FURNITURE. CAREFULLY TILT IT BACKWARDS. SWING THE FRONT CORNER SUPPORTS ROUND AND LOWER THE FURNITURE ON TO THEM. ENSURE THE CASTOR/ LEG IS LOCATED IN THE WELL OR THE FRAME/ BLOCK LEG IS RESTING ON THE TOP FACE. 7. ROTATE EACH CORNER SUPPORT ENSURING IT IS SNUG AGAINST THE FURNITURE. 8. RETIGHTEN THE WING NUT.



LARGE SOFAS WITH 6 CASTORS/ LEGS/ BLOCKS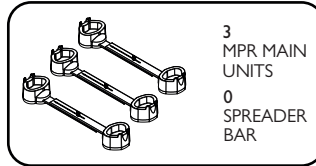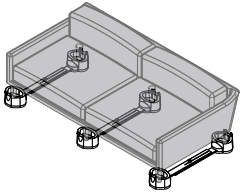

3
MPR MAIN
UNITS
0
SPREADER
BAR



MPR ASSEMBLY INSTRUCTIONS - LARGE SOFAS: 1. PLACE ONE CORNER SUPPORT UNDER EACH ARM, RUNNING FRONT TO BACK AND ONE MPR UNIT AS A CENTRAL SUPPORT IN THE MIDDLE. 2. IF THE FURNITURE HAS ANY ADDITIONAL POINTS OF CONTACT WITH THE FLOOR THEY TOO MUST BE SUPPORTED. 3. MAKE SURE EACH CORNER SUPPORT IS TIGHT UP TO THE FURNITURE AND TIGHTEN THE WING NUTS.

SINGLE BED/ TABLE WITH 4 CASTORS/ LEGS/ BLOCKS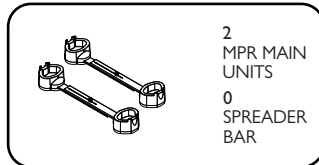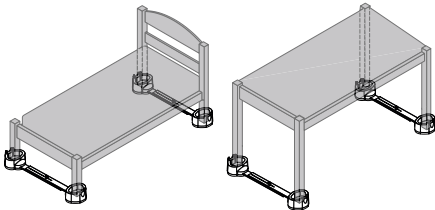



2
MPR MAIN
UNITS
0
SPREADER
BAR

DOUBLE BED WITH 6 CASTORS/ LEGS/ BLOCKS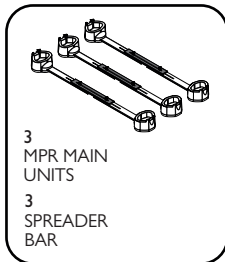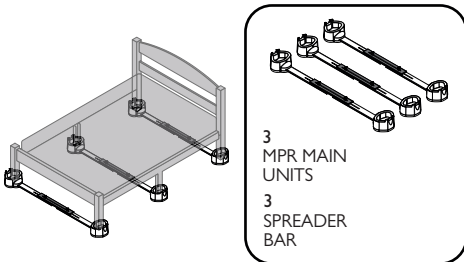

3
MPR MAIN
UNITS
3
SPREADER
BAR






DOUBLE DIVAN BED WITH 8 CASTORS/ LEGS/ BLOCKS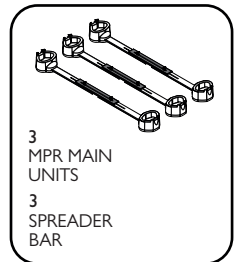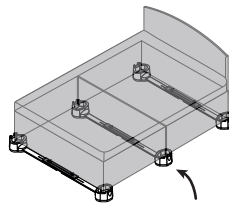

3
MPR MAIN
UNITS
3
SPREADER
BAR

MPR ASSEMBLY INSTRUCTIONS - BEDS AND TABLES: 1. PLACE ONE MPR UNIT PARALLEL TO THE FOOT OF THE FURNITURE. 2. LOOSEN WING NUTS AND ADJUST TO SAME WIDTH AS THE FURNITURE. FOR LARGER FURNITURE USE SPREADER LINK BARS TO EXTEND WIDTH. 3. LIFT FURNITURE AND PLACE CORNERS ON TO THE SUPPORTS. ENSURE THE CASTOR/ LEG IS SITTING INSIDE THE WELL OR THE FRAME/ BLOCK LEG IS RESTING ON THE TOP FACE. 4. ROTATE EACH CORNER SUPPORT ENSURING IT IS SNUG AGAINST THE FURNITURE. IF USING AN MPR AS A CENTRAL SUPPORT ENSURE IT IS PUSHED TIGHT TO SIDES OF FURNITURE. 5. REPEAT FOR THE OPPOSITE END OF THE FURNITURE. EVERY CASTOR/ LEG MUST BE SUPPORTED. 6. TIGHTEN ALL WING NUTS.







LEG TYPES AND MPR SUPPORT LOCATION

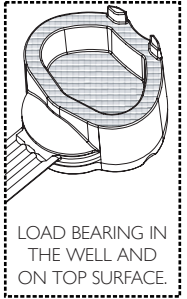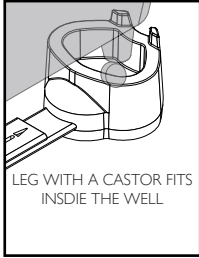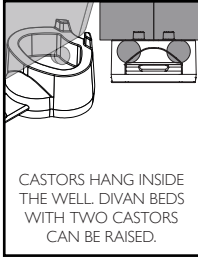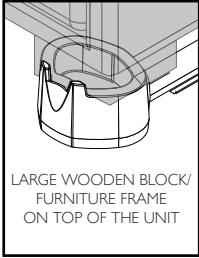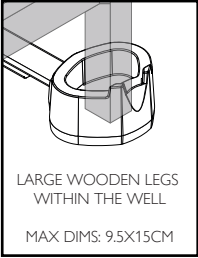
SURELOCK INSTRUCTIONS - OPTIONAL

ALWAYS REMOVE TAPED SURELOCK WASHERS FROM THE RAILS BEFORE USE OF THE FURNITURE RAISER. IF USING THE WASHER FOLLOW THE STEPS BELOW. ALTERNATIVELY, SET ASIDE AS USE THE RAISER AS NORMAL.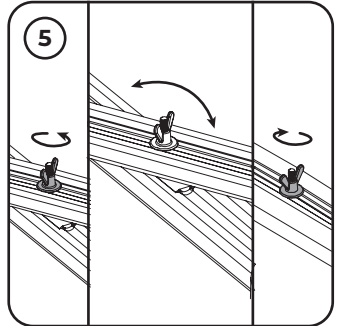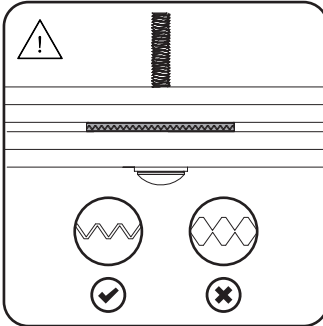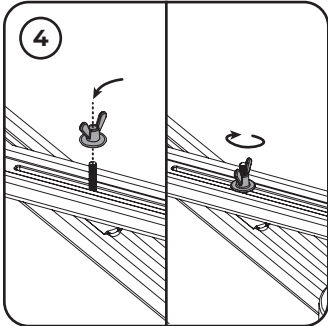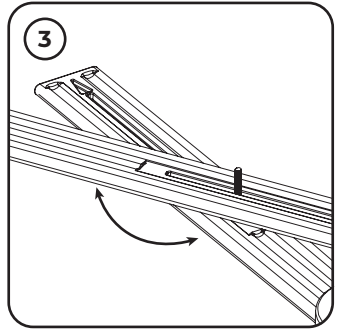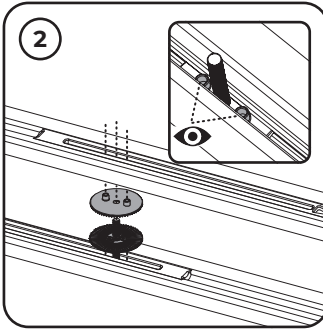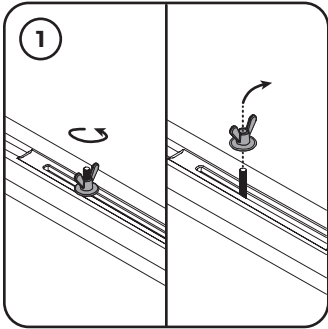

SURELOCK ASSEMBLY LOCATION

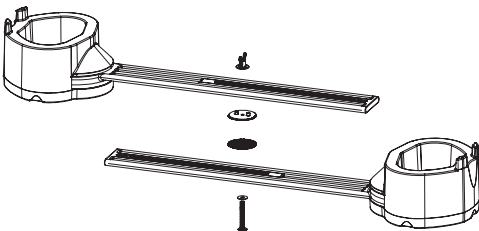SURELOCK INSTRUCTIONS: 1. FULLY UNSCREW WING NUT AND SET ASIDE. 2. INSERT THE TWO SURELOCK WASHERS WITH THE TWO TOOTHED SURFACES FACING EACH OTHER. ENSURE THE BOSSES CORRECTLY SIT WITHIN THE RAIL ADJUSTMENT SLOTS. 3. ALIGN AND POSITION THE FURNITURE RAISER AS REQUIRED. 4. WHEN IN THE DESIRED POSITION REATTACH THE WING NUT AND TIGHTEN. [!] WHEN POSITIONING THE RAISERS ENSURE THE TEETH OF THE LOWER SURELOCK SIT WITHIN THE TEETH OF THE UPPER SURELOCK. MISALIGNED TEETH OR TEETH NOT PROPERLY SEATED WILL CAUSE THE "LOCK" TO FAIL. 5. ENSURE THE WING NUT IS LOOSENED PRIOR TO ADJUSTING OR REPOSITIONING THE FURNITURE RAISER. THE WING NUT DOES NOT NEED TO BE FULLY REMOVED. THE WING NUT MUST BE RETIGHTENED FOLLOWING ANY ADJUSTMENTS.

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