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

Your BIG JOHN TOILET AID & COMMODE was specifically developed to provide a strong adjustable frame and backrest, combined with our big, comfortable BIG JOHN seat. Bringing the benefits of the BIG JOHN to be either used as a Toilet Aid, or a Commode which can be used anywhere. Hopefully making daily living that little bit easier.

Big John Commode Frame & Pan 66250






Big John Commode 66250/SET

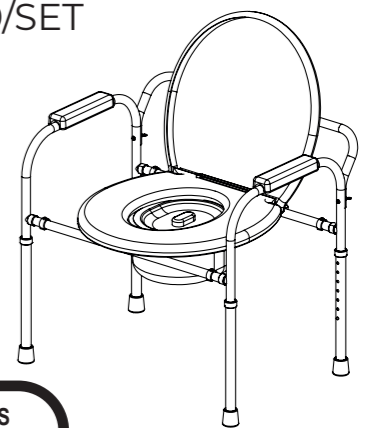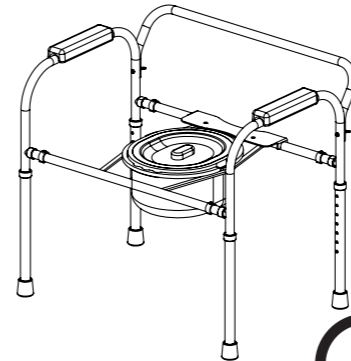





CAUTION: THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTIONS.

BHTA UK Healthcare Approved	7 ABS	7 NYLON 66	5 PP	Recycling icons	GORDON ELLIS & CO. TRENT LANE CASTLE DONINGTON DERBY, ENGLAND DE74 2AT	EC REP	AUTHORISED REPRESENTATIVE: ADVENA LTD. TOWER BUSINESS CENTRE, 2ND FLR, TOWER STREET, SWATAR, BKR 4013 MALTA
MD	IFU-REF 66250/0921-3		PART REF 66250 + 66250/SET		MADE IN THE USA		

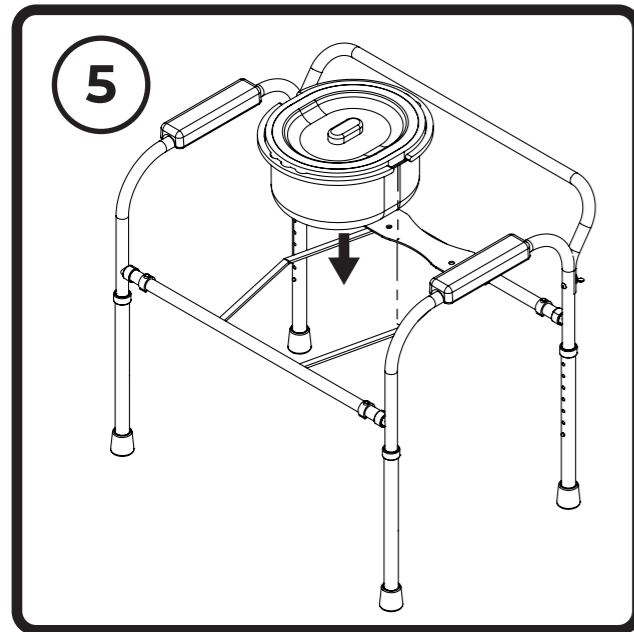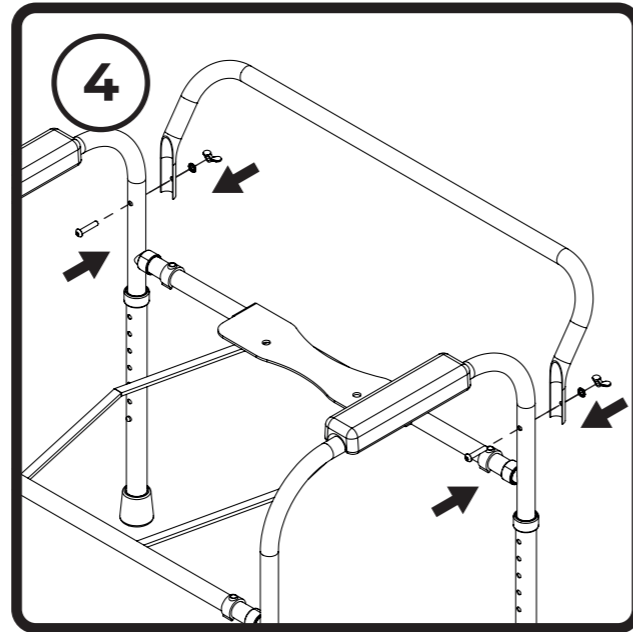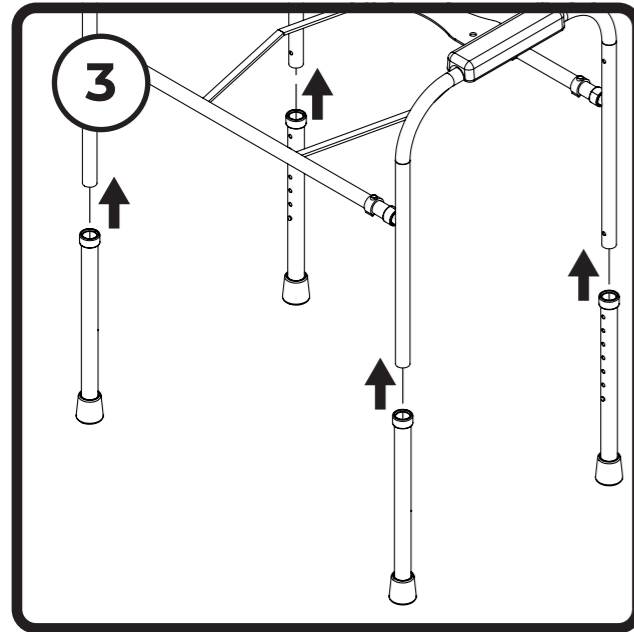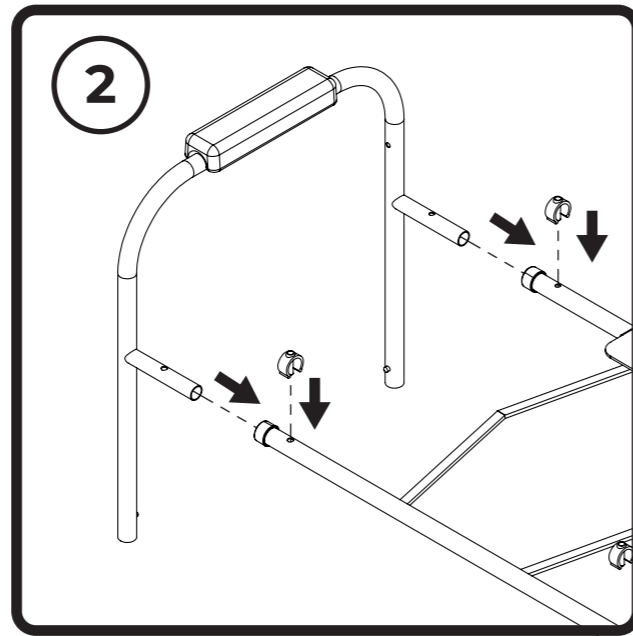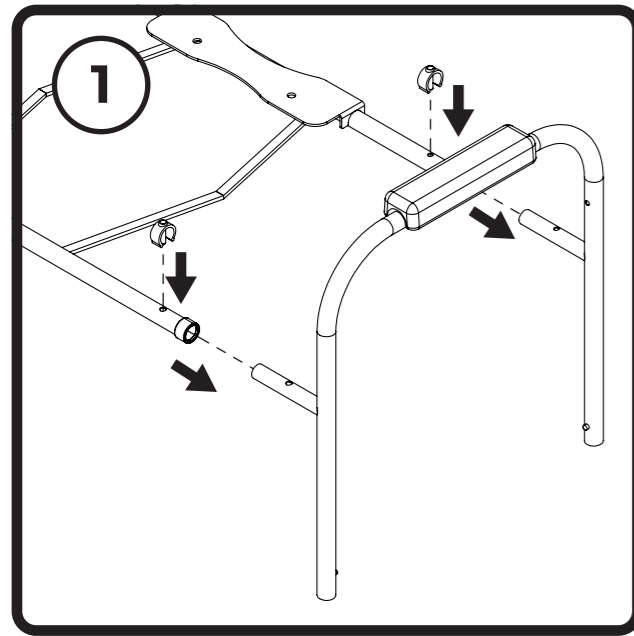1. ATTACH ONE SIDE ARM TO THE CROSS FRAME WITH THE PROVIDED CLIP, MAKING SURE THE CLIP IS FULLY INSERTED THROUGH THE HOLES.
2. REPEAT FOR THE SECOND SIDE ARM. ENSURE THE MOUNTING PLATE IS FACING UP AND TO THE REAR OF THE FRAME.
3. ATTACH THE LEGS.
4. ATTACH THE BACK REST USING THE BOLT, WASHER AND NUTS PROVIDED. ENSURE THE WASHER SITS FLAT ON THE FRAME.
5. INSERT BUCKET, IF REQUIRED. ENSURE HANDLES ARE OUT OF THE WAY.

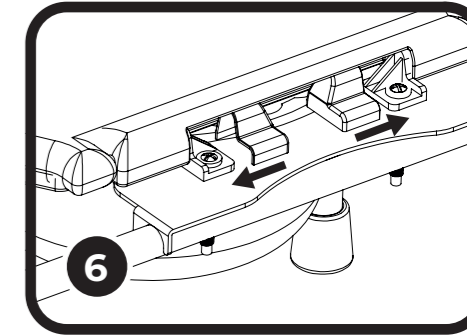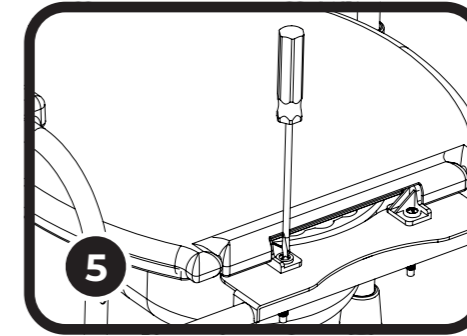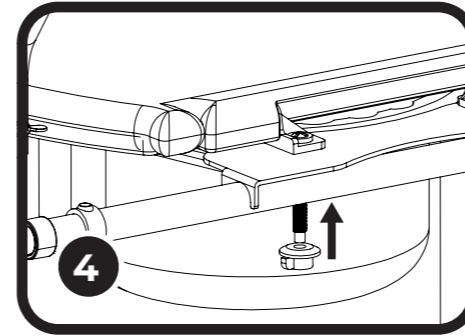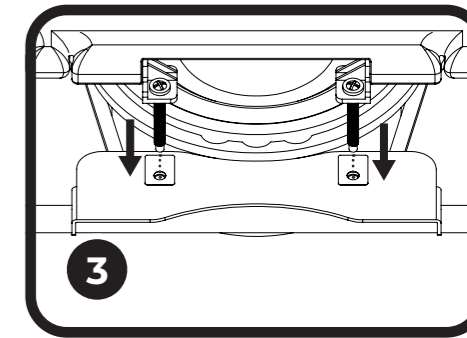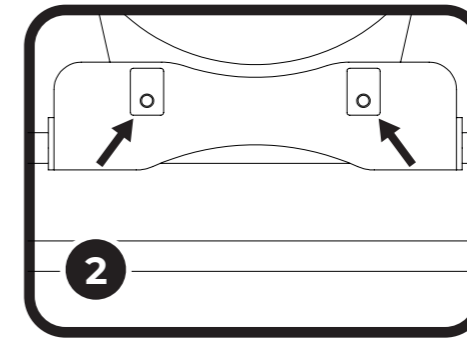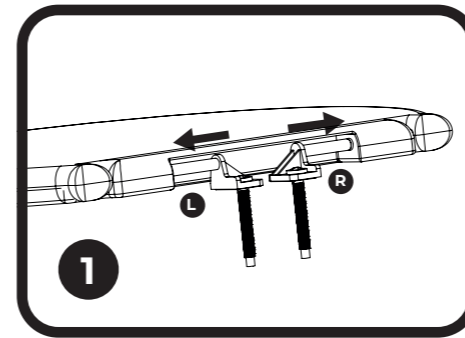1. INSERT THE HINGES THROUGH THE LEFT AND RIGHT SIDE TOILET SEAT AND LID.
2. LOCATE THE RUBBER WASHERS OVER THE HOLES ON THE SUPPORT BRACKET ON THE REAR CROSSBAR.
3. MOUNT THE TOILET SEAT BY LOCATING THE TWO HINGES THROUGH THE HOLES IN THE BRACKET.
4. FINGER TIGHTEN THE WING-NUTS ON TO THE BOLTS MAKING THE BEVEL SIDE IS FACING UP.
5. SECURE THE SEAT USING A SCREWDRIVER. DO NOT OVERTIGHTEN.
6. SLIDE AND CLIP BRACKET COVER INTO POSITION.

CHECK STABILITY BEFORE USE