# PRIMA® GRAB BAR

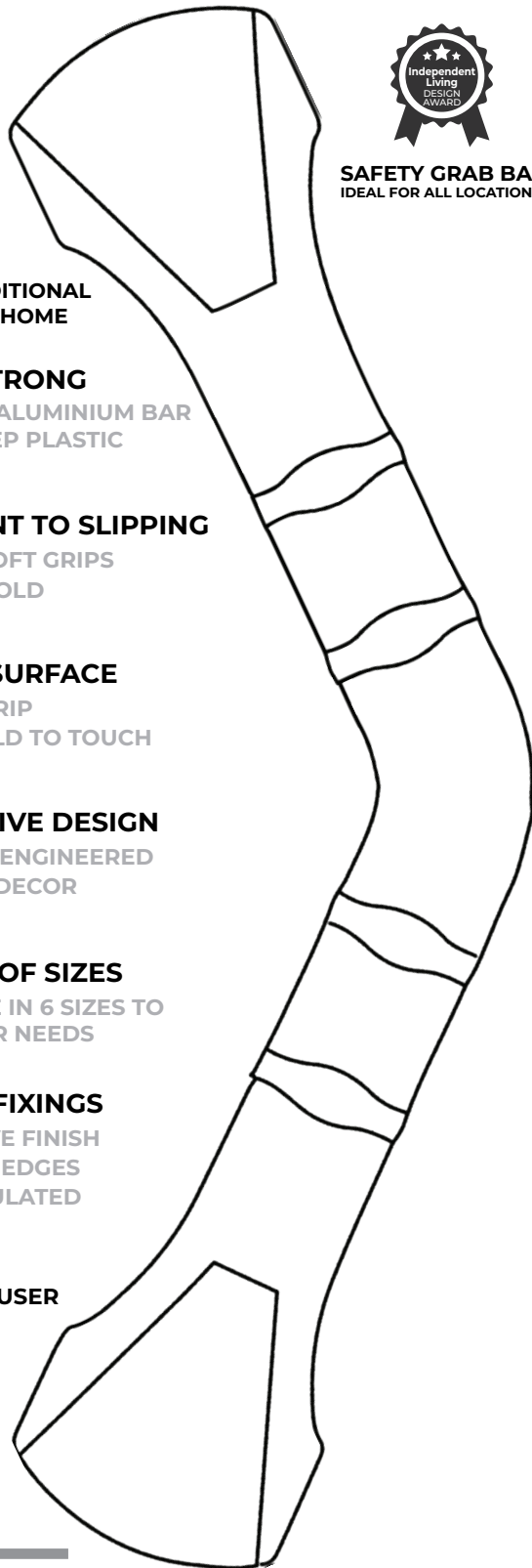
**SAFETY GRAB BAR**  
IDEAL FOR ALL LOCATIONS

**PROVIDES ADDITIONAL SAFETY IN THE HOME**

- **EXTRA STRONG**  
INTERNAL ALUMINIUM BAR  
EXTRA-DEEP PLASTIC
- **RESISTANT TO SLIPPING**  
SPECIAL SOFT GRIPS  
EASY TO HOLD
- **RIBBED SURFACE**  
EASY TO GRIP  
NEVER COLD TO TOUCH
- **ATTRACTIVE DESIGN**  
SUPERBLY ENGINEERED  
SUITS ALL DECOR
- **VARIETY OF SIZES**  
AVAILABLE IN 6 SIZES TO  
MEET YOUR NEEDS
- **HIDDEN FIXINGS**  
ATTRACTIVE FINISH  
NO SHARP EDGES  
FULLY INSULATED

⚠ **MAXIMUM USER WEIGHT**

**ANGLED**

A **PRIMA® GRAB BAR** FROM GORDON ELLIS IS AN INVALUABLE SAFETY AID FOR EVERYONE, ESPECIALLY THOSE WHO MAY BE UNSTEADY AND WOULD BENEFIT FROM A LITTLE EXTRA HELP

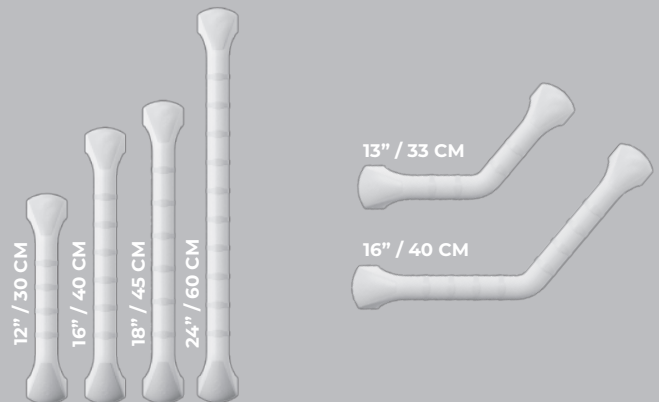
THE **PRIMA® GRAB BAR** CAN SERVE FOUR BASIC SAFETY FUNCTIONS:

- PUSHING UP
- PULLING UP
- STEADYING WHILE SITTING DOWN
- SECURITY WHILE TRANSFERRING POSITION

## THE PRIMA® GRAB BAR RANGE:

STRAIGHT

ANGLED

⚠ ALWAYS ENSURE THAT GRAB BARS ARE SECURELY FIXED TO THE WALL WITH SUITABLE FIXINGS. FIXINGS WILL VARY DEPENDENT ON WALL SURFACE. CONSULT A QUALIFIED PERSON IF NECESSARY. WALL FIXINGS NOT INCLUDED.

**SEE REVERSE FOR INSTRUCTIONS**  
AVAILABLE IN THE FOLLOWING LANGUAGES:  
EN FR DE IT ES PT JP NL EL DK SE NO FI

**GORDON ELLIS & CO.**  
TRENT LANE  
CASTLE DONINGTON  
DERBY, ENGLAND  
DE74 2AT

EC REP

**AUTHORISED REPRESENTATIVE:**  
ADVENA LTD. TOWER BUSINESS  
CENTRE, 2ND FLR, TOWER  
STREET, SWATAR, BKR 4013  
MALTA

MADE IN CHINA

MD

IFU-REF 6519/1120-3

PART REF 6519 / 6520  
6553 / 6554

VIEW THE ENTIRE RANGE ONLINE AT  
**GORDONELLISHEALTHCARE.CO.UK**

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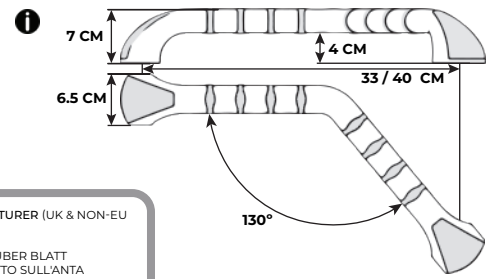
- EN**
1. Fitted correctly, angled rail must have one section horizontal to allow user extra leverage.
  2. The rail may be fitted to most types of solid wall. If fitting to plasterboard ensure suitable fittings are used. If in doubt consult a tradesman.
  3. Position the rail to a flat wall and mark the 4 screw positions.
  4. To fix in position use no. 10 x 50mm (3/16" x 2") roundhead woodscrews with suitable wall plugs. Depending upon application other fittings may be suitable.
  5. Locate the end caps to the rail and push down to fit.
  6. Before first use check the rail is securely fitted to the wall.
  7. Clean using non-scratch detergent.

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