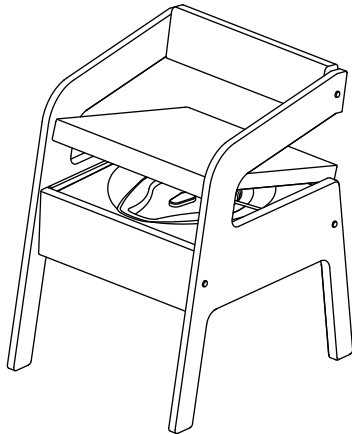
TIMBERRY SCANDI COMMODOE CHAIR

Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.

Your **TIMBERRY SCANDI COMMODOE CHAIR** was specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

STANDARD HEIGHT:

FLOOR TO CASSETTE HEIGHT: 43cm

LIGHT FRAME: 55211 - 55220

DARK FRAME: 55311 - 55320

WHITE FRAME: 55411 - 55420

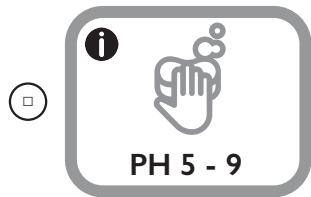
PETITE HEIGHT:

FLOOR TO CASSETTE HEIGHT: 39cm

LIGHT FRAME: 55211/P - 55220/P

DARK FRAME: 55311/P - 55320/P

WHITE FRAME: 55411/P - 55420/P

GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY, ENGLAND DE74 2AT

AUTHORISED REPRESENTATIVE
ADVENA LTD, TOWER BUSINESS
CENTRE, 2ND FLOOR, TOWER STREET,
SWATAR, BKR 4013 MALTA

MD

IFU-REF 55211/0724-1

REF

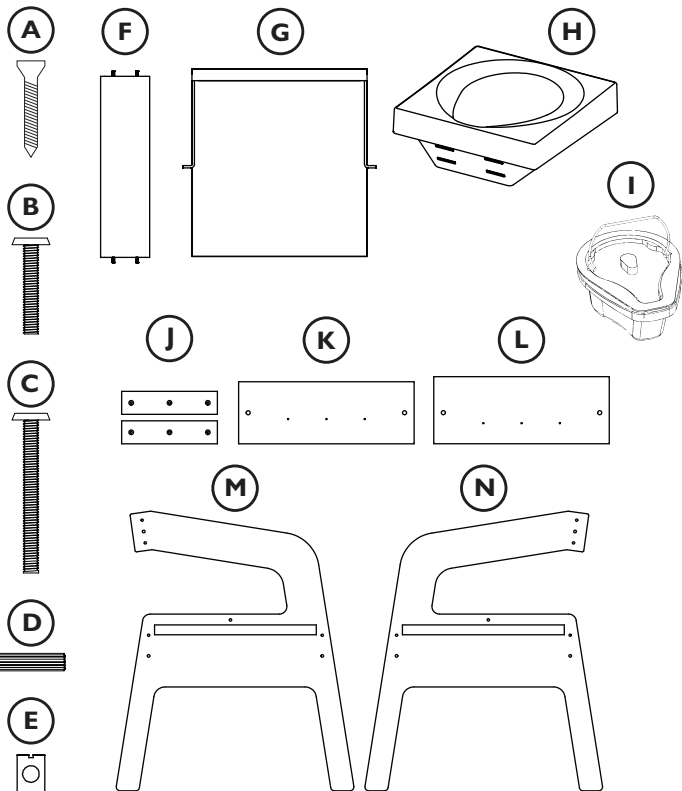
55211 TO 55220; 55311 TO 55320; 55411 TO 55420;
55211/P TO 55220/P; 55311/P TO 55320/P; 55411/P TO 55420/P

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

[gordonellis.healthcare](https://www.facebook.com/gordonellis.healthcare)

@EllisHealthcare

TOOLS

SUPPLIED:

4mm HEX KEY

REQUIRED:

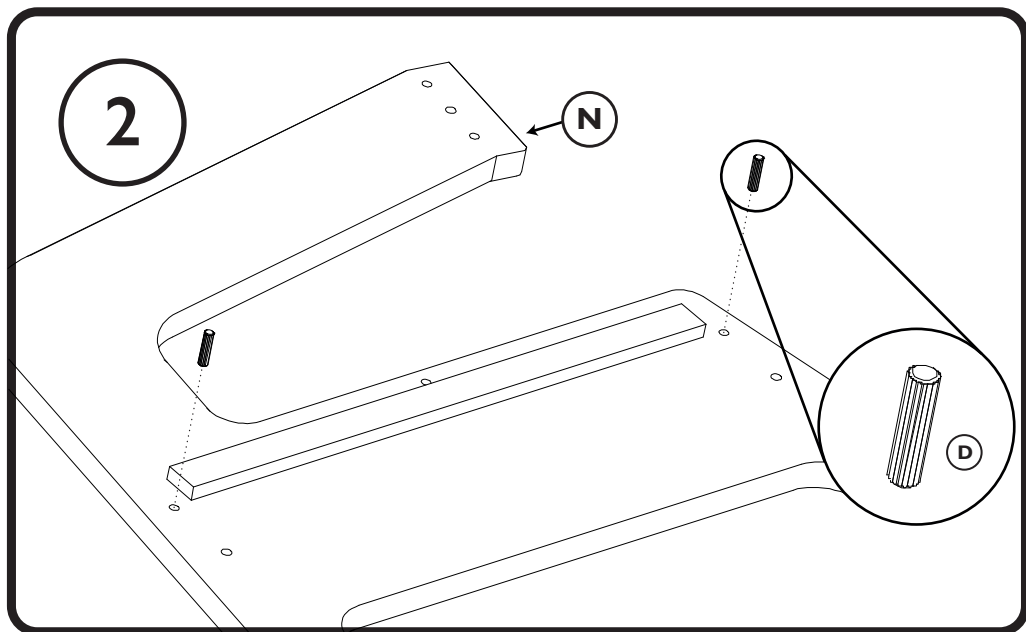
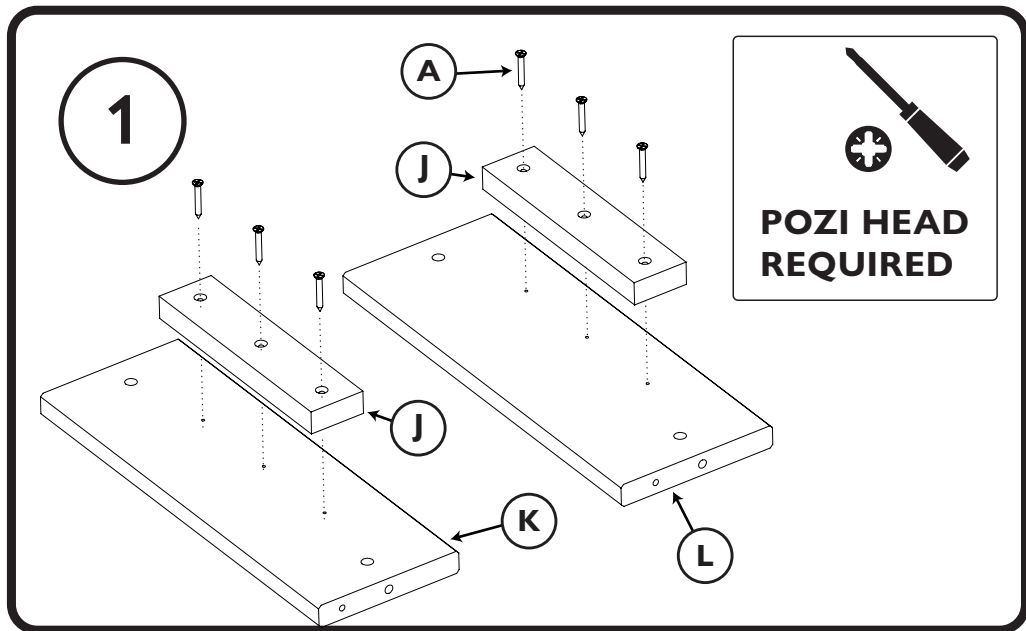
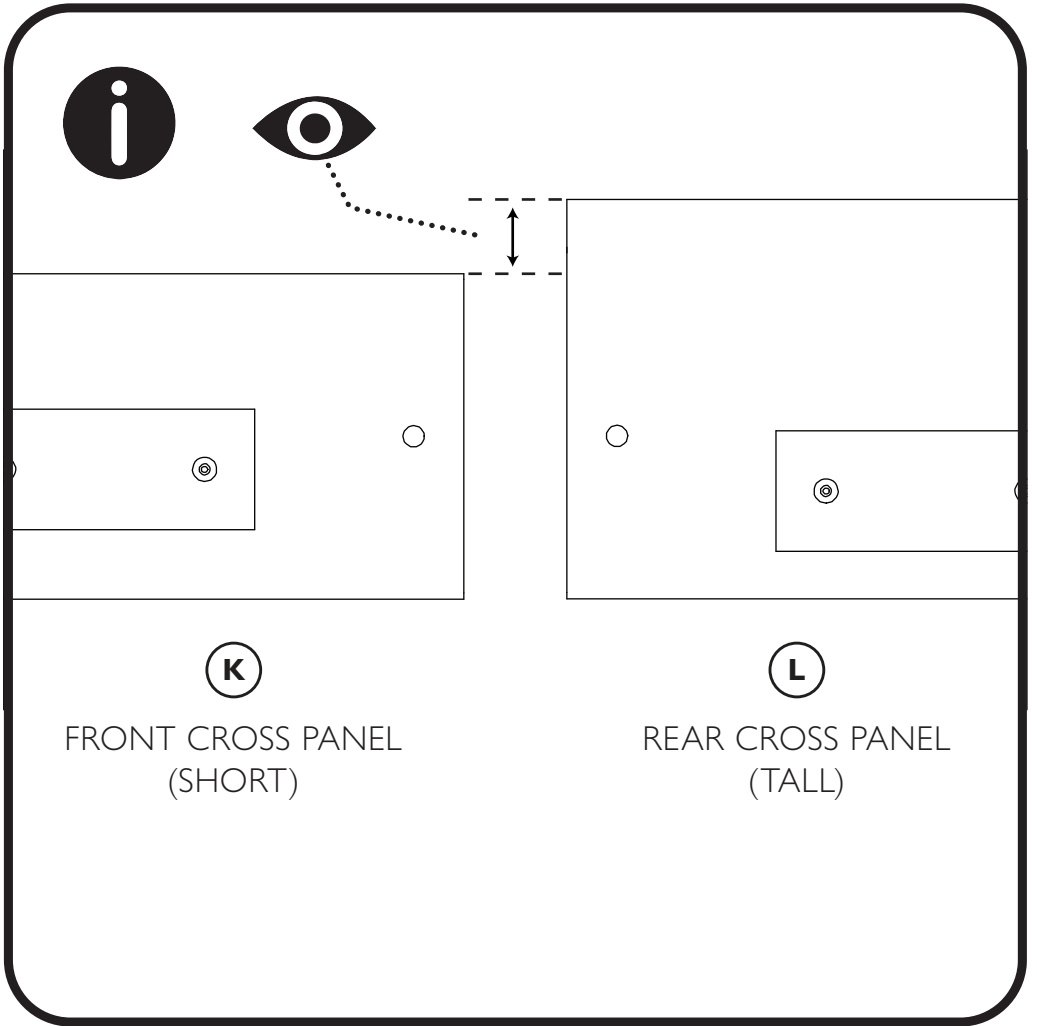
POZI-HEAD AND FLAT-HEAD SCREWDRIVER


| SUPPLIED PARTS: | QTY |
|----------------------|-----|
| A WOODSCREW | 6 |
| B M6X40 SCREW | 2 |
| C M6X60 SCREW | 4 |
| D WOOD DOWEL | 4 |
| E CROSS DOWEL | 4 |
| F BACK REST | 1 |
| G SEAT | 1 |



| SUPPLIED PARTS: | QTY |
|------------------------------------|-----|
| H SILVA CASSETTE | 1 |
| I ASHBY PAN AND LID | 1 |
| J SUPPORT RAILS | 2 |
| K FRONT CROSS PANEL (SHORT) | 1 |
| L REAR CROSS PANEL (TALL) | 1 |
| M A-FRAME (LEFT) | 1 |
| N A-FRAME (RIGHT) | 1 |

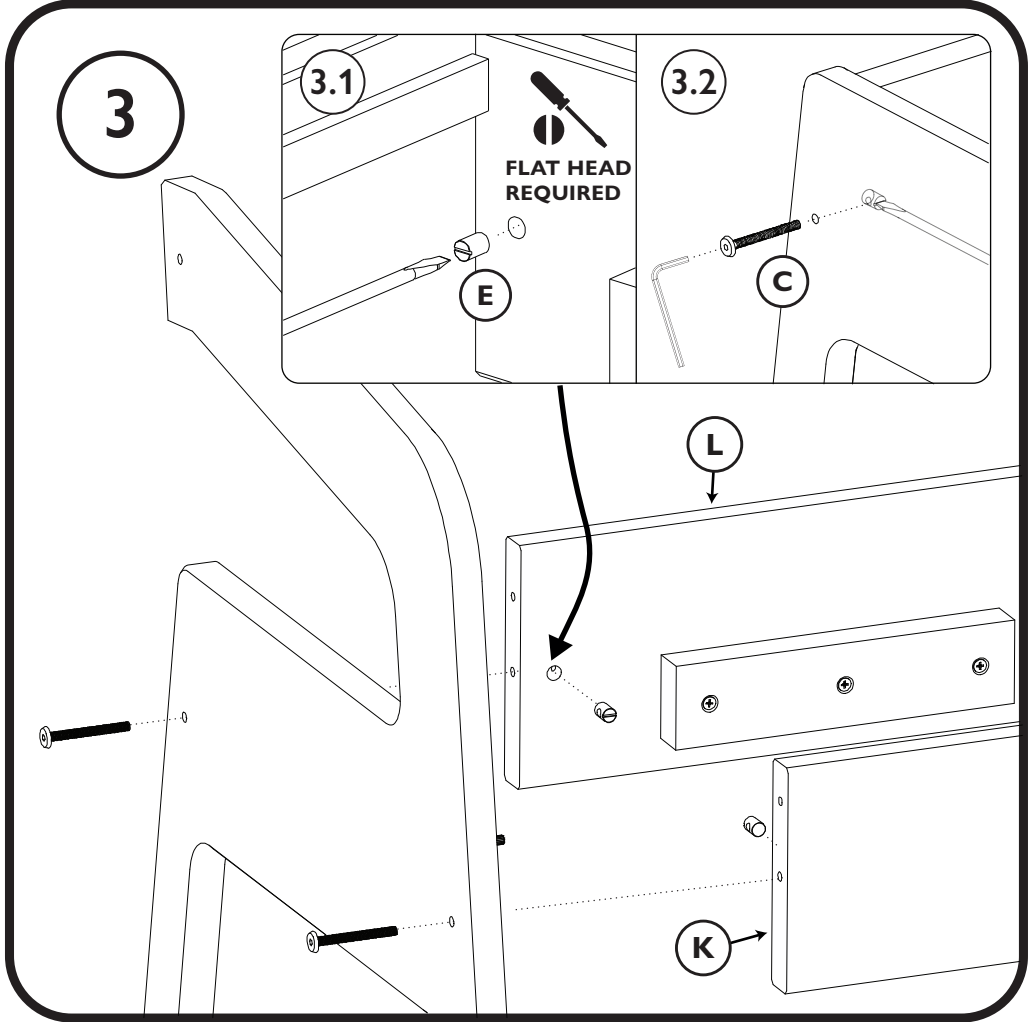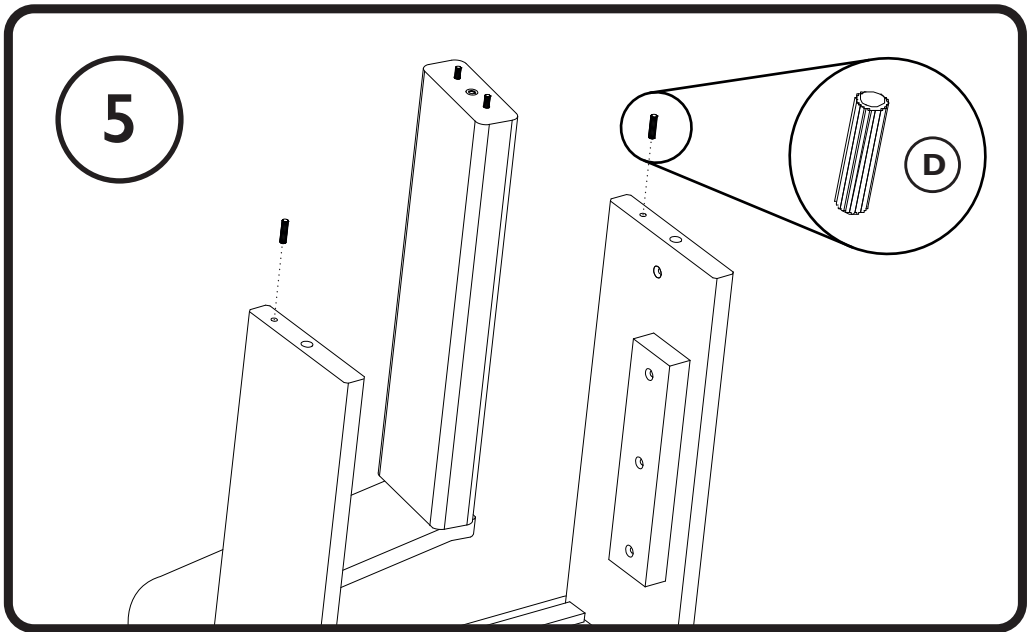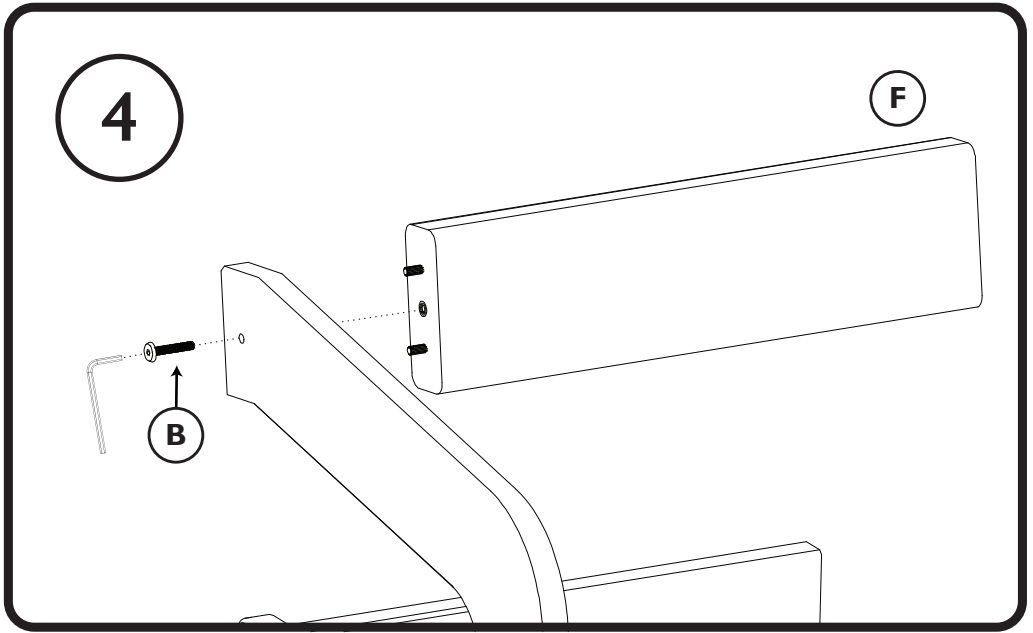❶ ASSEMBLY INSTRUCTIONS

6

3.1

FLAT HEAD
REQUIRED

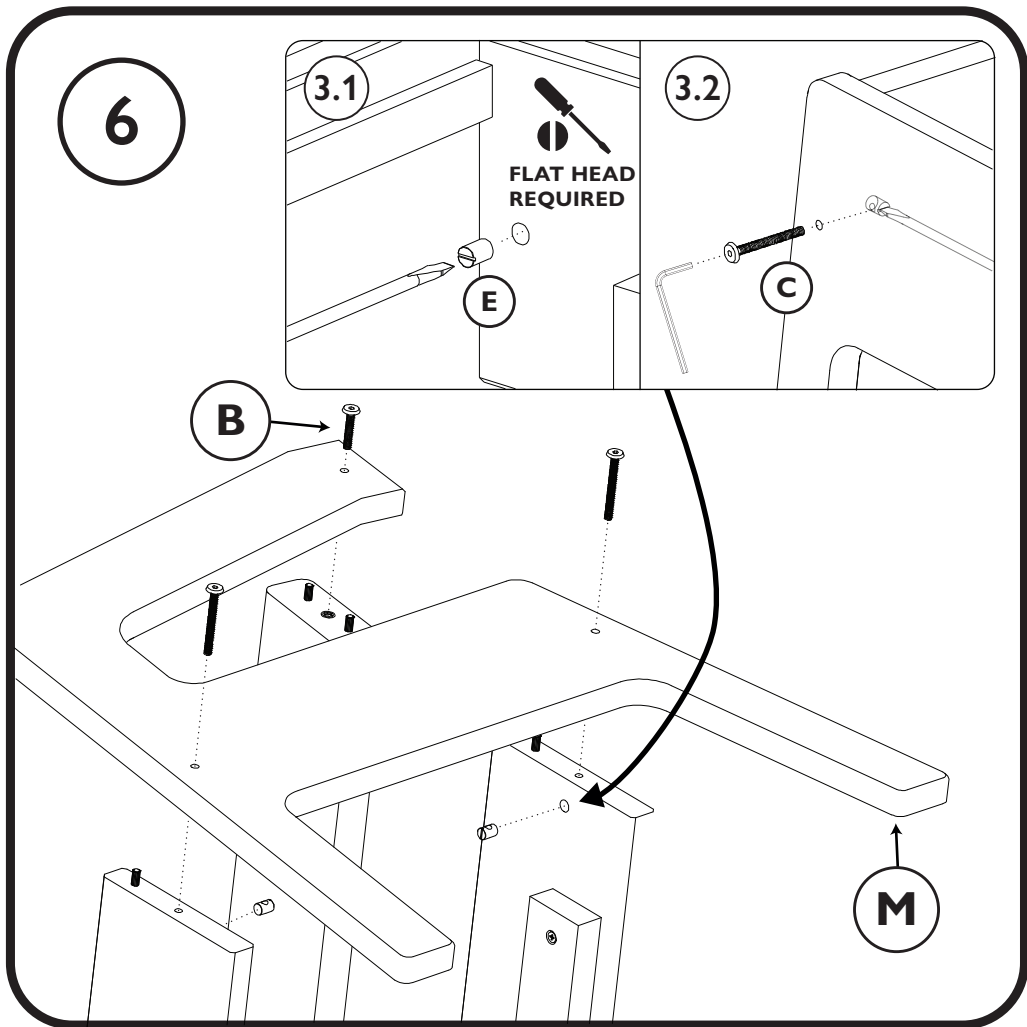
3.2

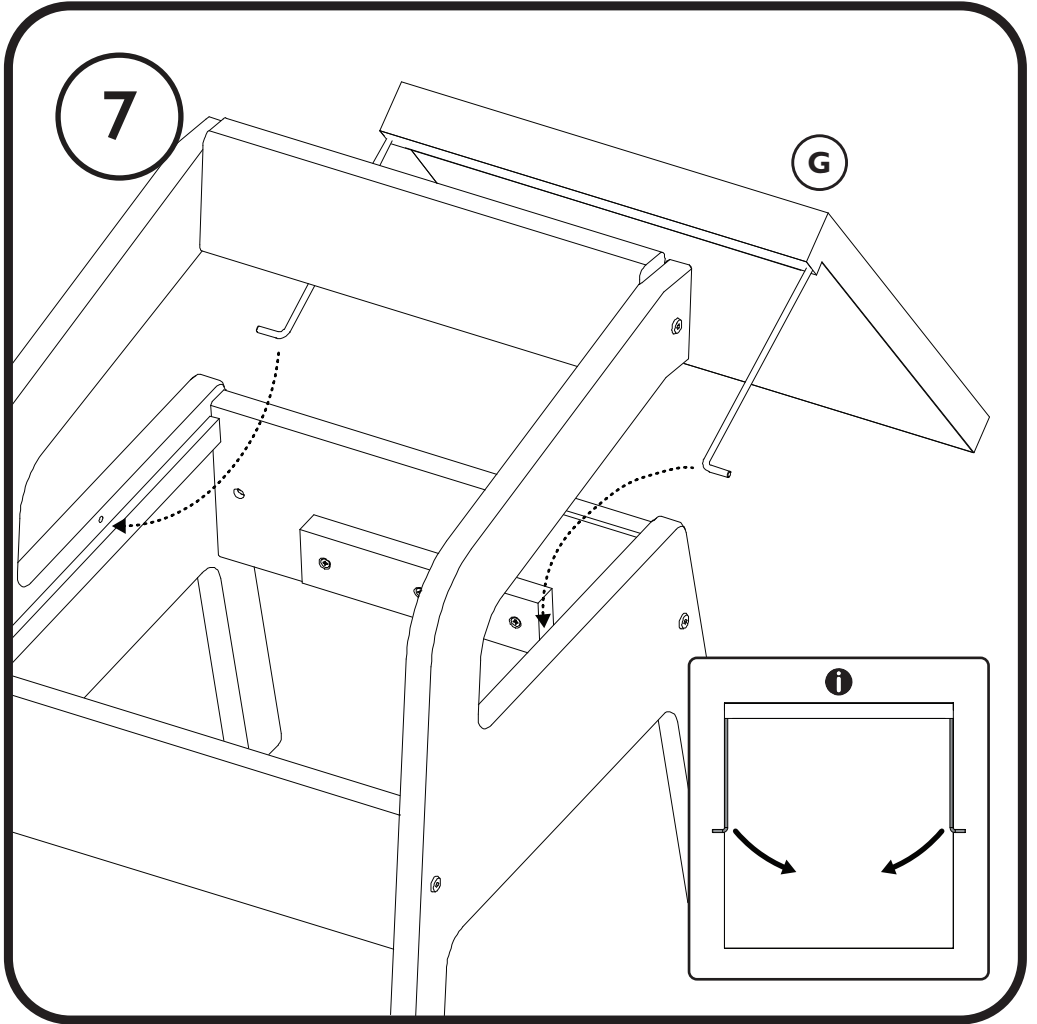
E

C

B


M




8

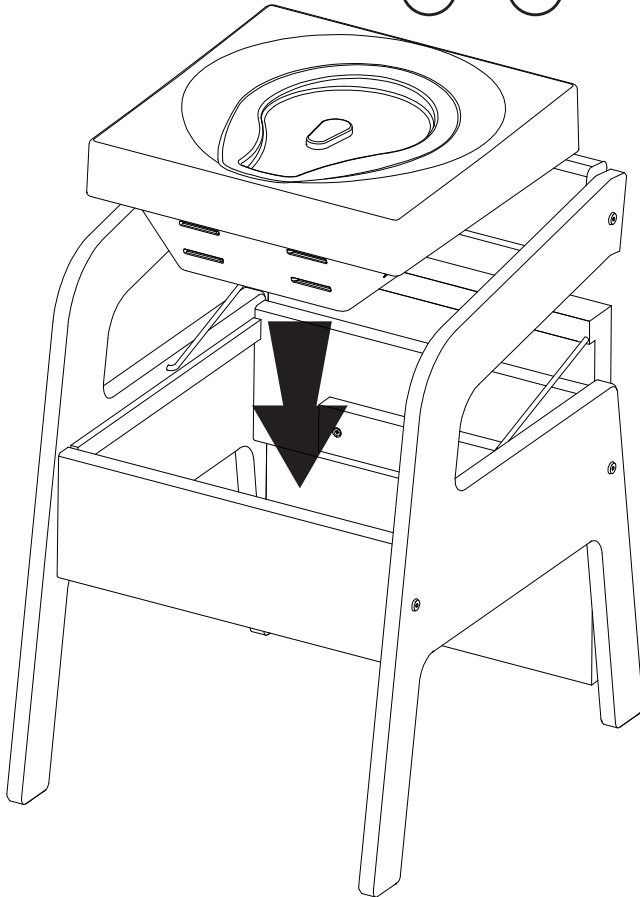H + I

i COMMODE INSTRUCTIONS

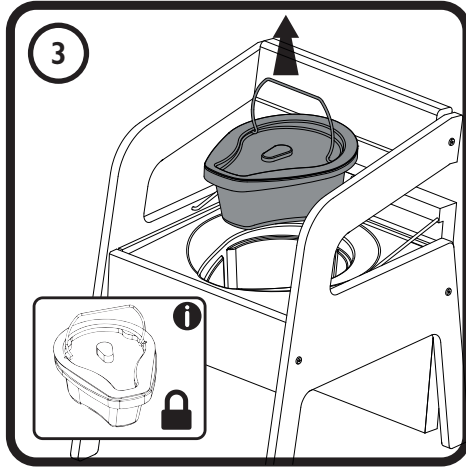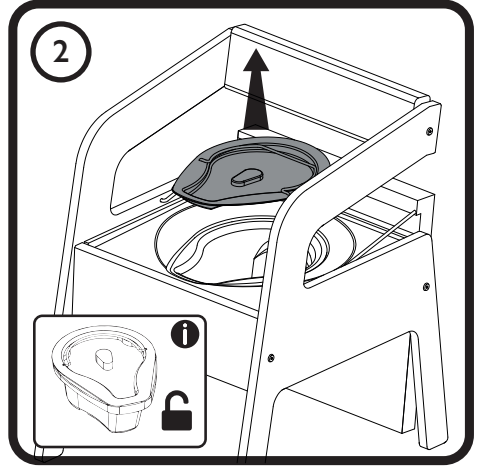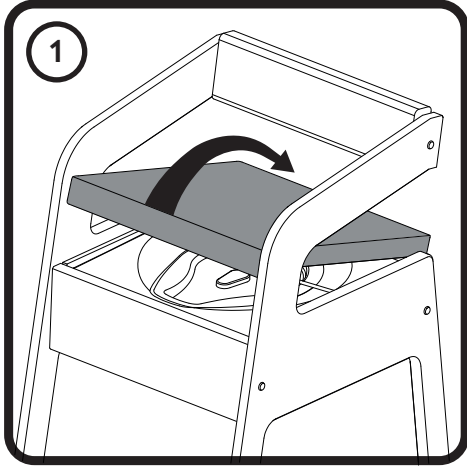ASSEMBLY INSTRUCTIONS: 1. SCREW THE RAILS TO THE PANELS USING A POZI HEAD SCREWDRIVER. 2. TURN ONE OF THE A-FRAMES ONTO ITS SIDE AND INSERT DOWELS. 3. ALIGN THE PANELS - ENSURE THE SMALLER PANEL IS AT THE FRONT WITH THE RAILS FACING INWARDS. INSERT THE METAL CROSS DOWEL USING A FLAT HEAD SCREWDRIVER. SECURE USING THE LONG SCREW. 4. ALIGN AND SECURE THE BACKREST USING THE SHORT SCREW. 5. INSERT THE REMAINING DOWELS. 6. ALIGN AND SECURE THE SECOND A-FRAME AS PER PREVIOUS INSTRUCTIONS. 7. ATTACH THE SEAT. FLEX THE METAL RAILS INWARDS AND ENSURE THEY SIT SECURELY INTO THE A-FRAME. 8. INSERT THE CASSETTE AND PAN.
COMMODE INSTRUCTIONS: 1. LIFT AND PIVOT THE SEAT PAD BEHIND THE CHAIR. 2. REMOVE THE LID FROM THE PAN - ENSURE THE HANDLE REMAINS DOWN TO ALLOW REMOVAL. USE THE COMMODE. 3. RETURN THE LID. LIFT THE PAN FROM THE COMMODE ENSURING THE HANDLE IS VERTICAL TO LOCK THE LID. FOLLOWING WASTE DISPOSAL, RETURN THE PAN AND SEAT PAD.

CHECK FOR STABILITY BEFORE EACH USE - STATIC USE ONLY. IT IS THE RESPONSIBILITY OF THE OWNER TO CARRY OUT AN INITIAL AND REGULAR RISK ASSESSMENTS. PERIODICALLY CHECK FIXINGS FOR TIGHTNESS AND ADJUST AS NECESSARY.

- ◇ = MAXIMUM USER WEIGHT.
- = WIPE WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.

ANY **SERIOUS ACCIDENT** OCCURRING THROUGH USE SHOULD BE REPORTED TO THE **EC REP** (EU CUSTOMERS) OR THE **MANUFACTURER** (UK & NON-EU CUSTOMERS)

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